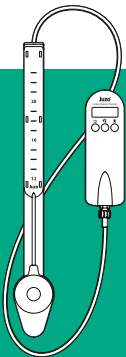
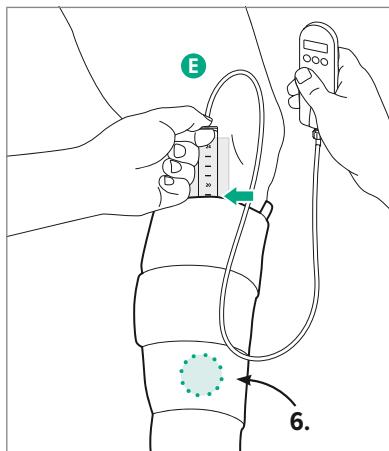
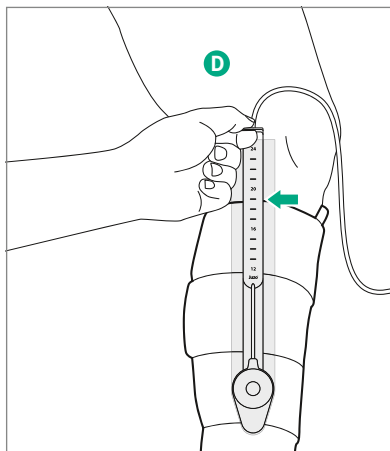
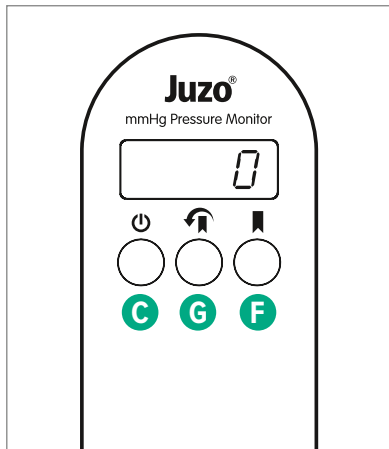
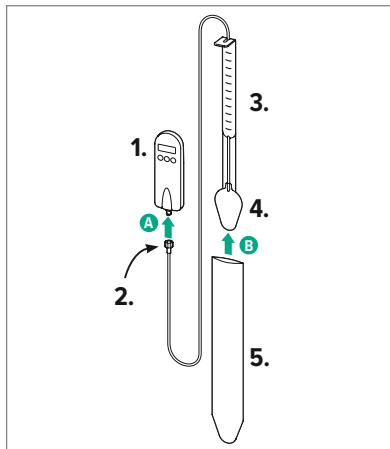
Juzo Pressure Monitor

Druckmessgerät
Pressure monitor

Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкция по эксплуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

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EN

Intended purpose

The Juzo Pressure Monitor has been designed to measure the compression on the leg exerted on a patient's extremity by a compression product (compression rolls or adjustable compression devices) that allows for variable pressure. This device is used to support and improve the treatment function of the compression products mentioned above. The Juzo Pressure Monitor has been designed to be used by specifically trained medical professionals only, who must follow the information in these instructions. The Juzo Pressure Monitor is not suitable for the pressure measurement of compression stockings.

Preliminary remarks

The Juzo Pressure Monitor is a device for measuring compression underneath compression rolls or adjustable compression devices. Thanks to the dynamic display, it can be used as a guide showing the resting as well as the working pressure. A specially developed positioning aid means that measurements can be taken under compression rolls or systems that have already been applied. The device-dependent tolerance range of the measured values is +/- 3 mmHg. The pressure display remains dynamic during the measurement, meaning that no static value is shown. In addition to this, surrounding factors

(e.g. muscle activity, placement of the pressure sensor on bony protrusions, shape and circumference of the extremity, elasticity of the compression material) have a direct impact on the pressure values displayed. This means that the measured values are constantly fluctuating. They can therefore merely be seen as a guide. Thus, measurement of compression using the Juzo Pressure Monitor must never replace observation of the patient (e.g. mention of pain, noticeably pale and/or cold extremity).

Juzo Pressure Monitor contents

- 1 hard case
- 1 monitor and control unit
- 1 sensor unit (consisting of Luer-Lock connecting hose, pressure sensor, positioning aid)
- 1 pack containing 50 disposable protective covers (more can be ordered)
- 1 Instructions for use

Important information

Use of batteries

- Do not use a different type of battery for this device than indicated in the battery compartment.
- Do not insert the battery in the wrong direction (poles on the wrong side).
- A flat battery must be immediately replaced with a new one.
- Remove the battery if the device will not be used for three months or more.
- If battery acid makes contact with your skin, rinse it immediately with a lot of clean water. Make sure you avoid contact with your eyes. In the case of contact, consult a doctor immediately!

General use

- The device has been exclusively designed to measure the pressure of compression rolls or adjustable compression devices. Do not exert excessive pressure when inserting the positioning aid under compression garments. The Juzo Pressure Monitor is not suitable for the pressure measurement of compression stockings.
- The Juzo Pressure Monitor has been designed to be used by specifically trained medical professionals only.
- Compression measurement must never replace patient observation (e.g. mention of pain, pale and/or cold extremity).
- Do not leave the device unattended within the reach of small children.
- Use the sensor unit on undamaged skin only.
- Do not exert excessive pressure when inserting the positioning aid under compression garments.
- Do not take the control unit apart.

The Luer-Lock connecting hose, pressure sensor and positioning aid make up the sensor unit. Do not take them apart! The Luer-Lock connector has been exclusively designed to connect the control unit with the sensor unit. The disposable protective covers have been exclusively designed for the sensor unit. They protect the pressure sensor and help with hygiene. Use a new protective cover for each patient.

The Juzo Pressure Monitor has not been designed for long-term patient measurements. The device was calibrated before delivery and should not require maintenance for its usage period of 5 years. You are welcome to return waste equipment to us free of charge. For more information, please contact our Juzo Customer Service.

General warnings

Ensure that the sensor unit is not damaged by pointed objects or those with sharp edges. Any damage to the control or sensor unit (in particular the pressure sensor) can have a direct impact on the functionality of the device. Only ever use the device if it is in perfect condition. Before disposing of the device or its accessories, read and follow the section "Storage, cleaning and disposal".

Elements and positioning of the Juzo Pressure Monitor and control unit

1. Monitor and control unit
2. Luer-Lock connecting hose
3. Positioning aid
4. Pressure sensor
5. Disposable protective cover
6. Correct placement of the pressure sensor for measurement

Preparing the device for use

- A.** Before first use, please insert an appropriate battery. Please note the battery requirements in the battery compartment. Then connect the Luer-Lock screw cap at the end of the air hose with the control unit. Ensure that the connection has been screwed on tightly and cannot become loose by itself.
- B.** Insert the sensor unit into a disposable protective cover. Do not use the Juzo Pressure Monitor without a protective cover. In addition to hygienic reasons, it also protects the pressure sensor from possible snapping off or slipping.

Using the Juzo Pressure Monitor

- C.** Press the Power button  to turn the Juzo Pressure Monitor on. Initially, the display will show "888" while the device automatically resets. Wait about 3–4 seconds until the display

shows **"0"**. If you press the Power button again, the device will turn off. The device will turn off automatically if there is no activity for 5 minutes.

- D.** Place the pressure sensor medially on measurement point B1 (calf muscle insertion point), which is about 10 – 15 cm above the medial malleolus. The scale of the positioning aid must point outwards. In order to determine correct placement of the pressure sensor, first hold the positioning aid **over** the applied compression bandages (or the applied compression rolls). Read on the positioning aid how far under the compression garment the pressure sensor needs to be inserted to be located in the correct position.
- E.** Now, using the positioning aid, ease the sensor unit **under** the compression garment until the centimetre value previously determined is displayed at the upper edge of the compression garment. Apply gentle but not excessive pressure on the extremity as you do this to make the insertion easier. The dynamic display shows pressure values from 1 to 154 mmHg. Pressure values above 155 mmHg are displayed as "HI".
- F.** Press the Save button  to save a reading. As confirmation, the display will flash once, then immediately return to the dynamic display. You can save 10 readings.
- G.** You can retrieve saved readings by pressing the  button. The last saved reading will be displayed first. To return to the dynamic display, press the Power button .

If "LOW" is displayed, the battery is flat and must be replaced. The battery compartment is located on the back of the monitor and control unit. Push the cover of the battery compartment in the direction of the arrow to open it. Remove the flat battery and insert the new battery in accordance with the indication of plus and minus pole inside the compartment. Do not use a different type of battery than indicated in the battery compartment. Please follow the instructions about disposing of the old battery.

Important: In connection with an adjustable compression device, we generally recommend using the sensor unit over a liner stocking to make inserting and removing of the positioning aid easier. In particular with very tight compression rolls, excessive sweating and/or very thin, sensitive skin, the sensor unit should be positioned before putting on the compression rolls to prevent damaging the skin. To make this process easier, you can secure the positioning aid under the tubular bandage. To remove the sensor unit from under the compression garment, pull it out from the upper end of the positioning aid. For this, we recommend gently pushing the positioning aid against the extremity without applying excessive pressure. At the same

time, you should hold on to the top of the connecting hose to prevent the pressure sensor from slipping off the positioning aid. If it is difficult to remove the sensor unit from under the compression garment (e.g. in the case of excessive sweating), pull it out without the protective cover first, then carefully remove the protective cover.

Storage and cleaning

With the exception of the battery, the device has been designed for a usage period of 5 years. Store the device at room temperature and protect it from sunlight, humidity and strong vibrations (do not bump it or drop it). When transporting the device or not using it, only ever store it in the designated hard case. This is the only way you can guarantee that the measuring unit is not excessively bent, stressed or overtightened. Store the device away from small children. Remove the battery if the device will not be used for three months or more. Do not use any solvents for cleaning the device. If required, clean the monitor and control unit with a soft, dry cloth. If required, clean the sensor unit with an appropriate agent for wipe disinfection.

Indications

Measuring contact pressure of compression wraps, measuring contact pressure under adjustable compression devices (ACD).

Contraindications

Open wounds, untreated, advanced skin conditions and contact dermatitis, contact allergy to PET or plastic

Relative contraindications

Pressure sensitivity and sensitivity disorders

Important

If there are open wounds in the area to be treated, the affected area must be covered with an appropriate wound dressing or bandage before the compression garment can be applied. Liability for failure to comply with these contraindications cannot be accepted by the Julius Zorn GmbH Group.

Warranty and reporting of serious incidents:

Statutory warranty regulations apply. In the event of complaints related to this product, such as damage to the material, please contact our Customer Service Team. Serious incidents that may lead to a major deterioration in the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) of Regulation (EU) 2017/745 (MDR).

Disposal

This electric or electronic device must **not** be disposed of with household waste. Please follow regional regulations for the disposal of electronic waste and dispose of old appliances in the proper way. You are responsible for deleting personal data from the device.

Do not dispose of the battery that you used in the device with household waste; take it to a battery collection box at a retail outlet. Before disposal, tape up the poles to prevent short circuiting.

Dispose of used protective covers with household waste. Please follow regional regulations for the disposal of plastic waste.

Disposal regulations may vary depending on your country. Therefore, please follow the national guidelines in the relevant country.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com
Berro AG · Austrasse 7 · 9055 Bühler · Suisse
UK responsible person: Juzo UK Ltd